


Ready To Ship In 4 Easy Steps

1

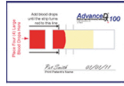

Please place completed blood sample card inside the foil return bag.
BE SURE YOU HAVE WRITTEN YOUR NAME AND DATE OF BIRTH ON THE CARD

2



Insert the foil return bag that now contains the completed blood sample card into the clear biohazard bag. Then, place it inside the internal testing materials box.

3



Place the internal testing materials box that now contains the blood sample card and the biohazard bag, inside the prepaid shipping envelope.

4



Drop your sample in the mail!



Once your test has been processed, you will receive a notification that your results are completed and available via CashLabs.com or our mobile app.