


IMPORTANT

Before collecting your sample, please register your kit at: CashLabs.com/Register. Your Test Kit I.D. can be found on the Test Kit I.D. Card contained your box.

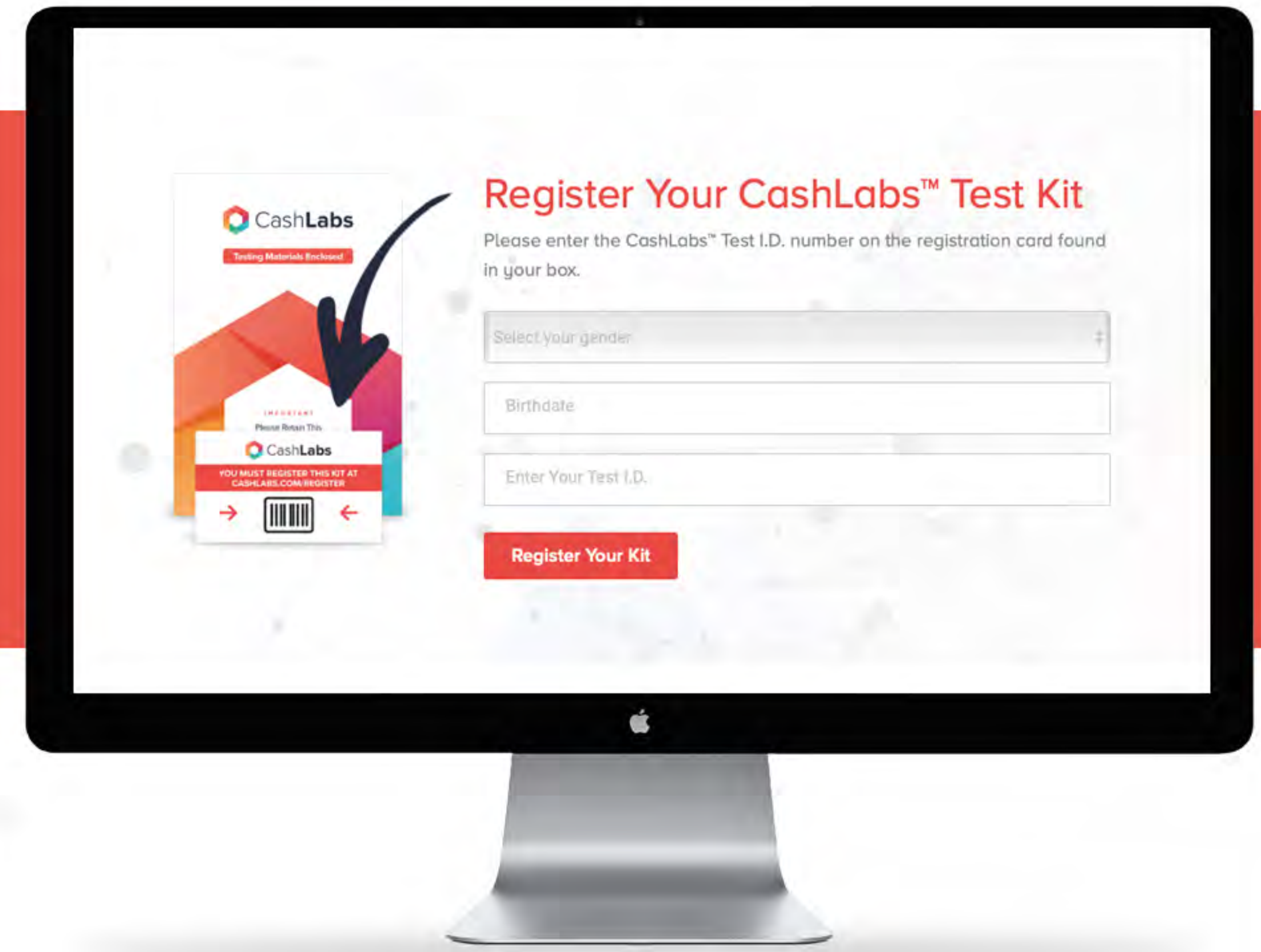

Preparation Instructions

Run your finger under warm water to increase the blood flow. Let your hand hang for a few seconds to increase the blood to that finger.

Sample Collection Instructions *(All Food Sensitivity Tests)*



Step 1: Identify the desired puncture site (shaded area) and make sure it is properly cleansed with warm water.



Step 2: Twist off tab and discard.



Step 3: Position safety lancet firmly against puncture site as illustrated. Hold safety lancet between fingers.



Step 4: To activate, press safety lancet firmly against the puncture site. Do not remove the device from the site until an audible click is heard.



Step 5: Allow blood to drip on to the Protein card. Please make sure you fill three of the five circles.



Step 6: Apply a bandaid to the area that is bleeding.