


## IMPORTANT

Before collecting your sample, please register your kit at: [CashLabs.com/Register](https://CashLabs.com/Register). Your Test Kit I.D. can be found on the Test Kit I.D. Card contained your box.

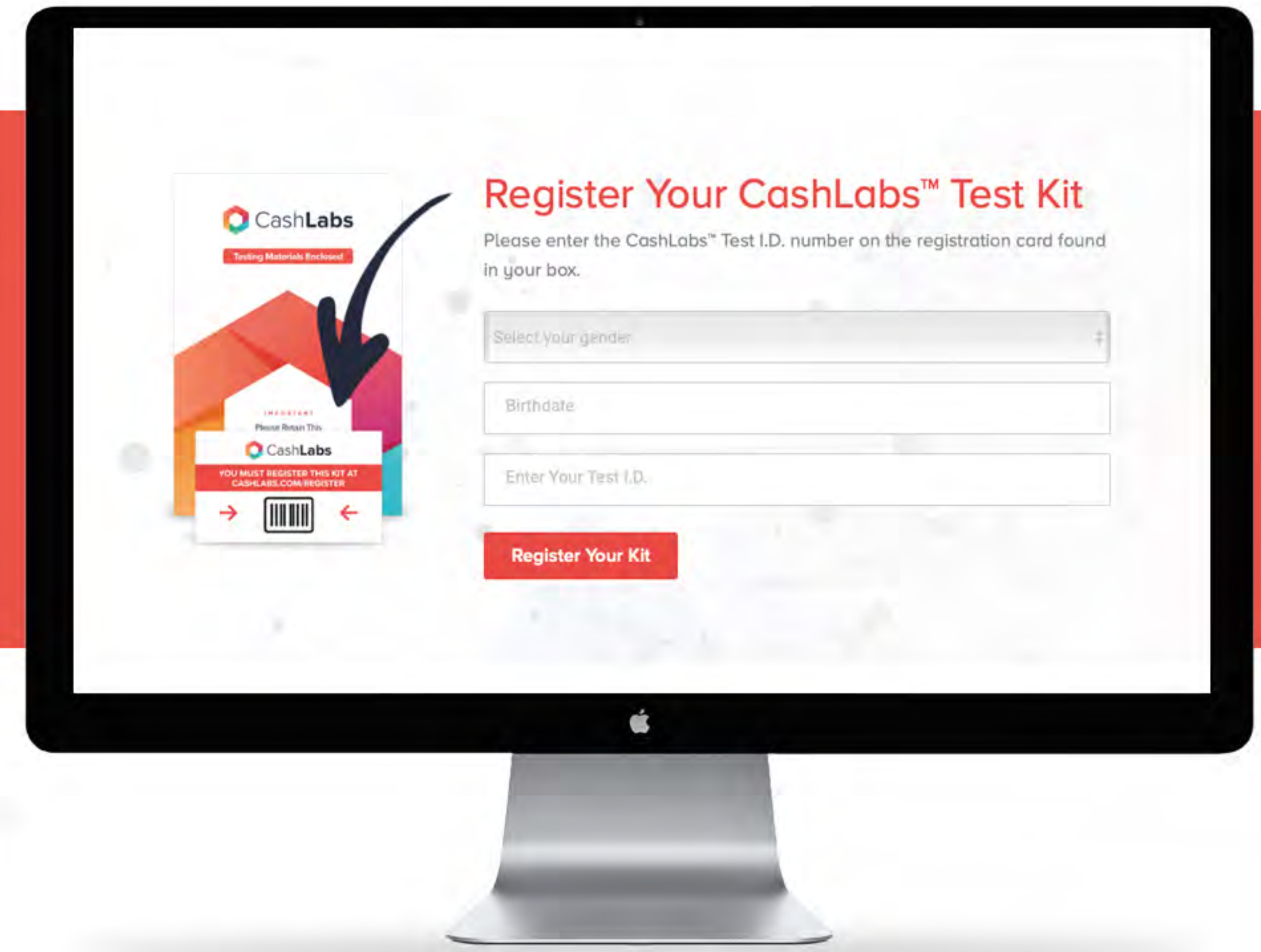

## Preparation Instructions

Run your finger under warm water to increase the blood flow. Let your hand hang for a few seconds to increase the blood to that finger.

## Sample Collection Instructions *(excluding Food Sensitivity Tests)*



**Step 1:** Identify the desired puncture site (shaded area) and make sure it is properly cleansed with warm water.




**Step 2:** Twist off tab and discard.



**Step 3:** Position safety lancet firmly against puncture site as illustrated. Hold safety lancet between fingers.



**Step 4:** To activate, press safety lancet firmly against the puncture site. Do not remove the device from the site until an audible click is heard.

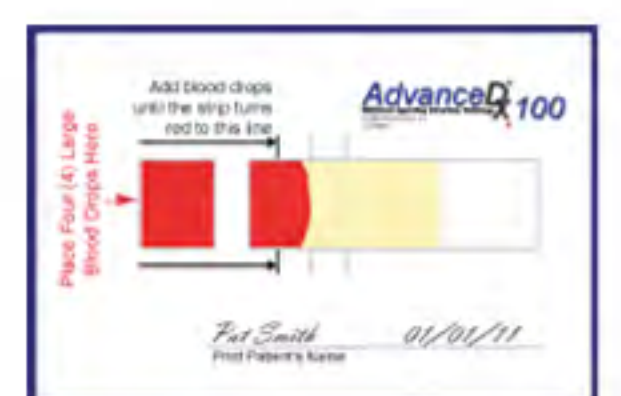

**Step 5:** Disperse large drops of blood into the collection square until the line indicated on the card is reached.



**Step 6:** Apply a bandaid to the area that is bleeding.